

meet.amber360.com/#/dashboard/welcome

Traxbean Dashboard Business Monitor Alarms System

Workbench x

Hello Demouser, Your local time is 2024-10-25 08:08:35

**Get Started**

Dear user, welcome to our smart wearable products and IoT devices family. If you are using it for the first time, please click the button below and we will provide you with a comprehensive system tour service. Through this process, you will quickly understand how to operate and make full use of our products. If you encounter any problems during the operation, please feel free to contact our sales consultant or technical service team at any time, and we will be happy to assist you.

Let's Get Started

Heat Map

Alarms

2024 Oct Month Year

Su	Mo	Tu	We	Th	Fr	Sa
29	30	01	02	03	04	05
06	07	08	09	10	11	12
13	14	15	16	17	18	19

1. Choose 'Business'

Traxbean Dashboard Business Monitor Alarms System

Workbench x

Persons Targets

App User List Guardian

Devices Close Cases

Tasks Person List

Let's Get Started

Heat Map

Alarms

2024 Oct Month Year

Su	Mo	Tu	We	Th	Fr	Sa
29	30	01	02	03	04	05
06	07	08	09	10	11	12
13	14	15	16	17	18	19

2. Choose 'Target'.

The person with a wearable device forms a target.

Traxbean Dashboard Business Monitor Alarms System

Workbench Targets x


Keyword: Please enter Keyword Department: Please choose Depa... Reset Search

Target List


Send Message Add Tag Add Export

<input type="checkbox"/>	Name	IMEI	Gender	Age	Phone	Department	Tags	Menu
<input type="checkbox"/>	MT4 77353	355932600077353	Male	0		Demo	—	Menu
<input type="checkbox"/>	TEST MAPTS	35593260008788	Male	0		Demo	—	Menu
<input type="checkbox"/>	MT4 65135	355932600065135	Male	0		Demo	—	Menu
<input type="checkbox"/>	MT4 65127	355932600065127	Male	0		Demo	—	Menu
<input type="checkbox"/>	kim yang	355932600065119	Male	9		Demo	—	Menu
<input type="checkbox"/>	kim yang	35593260059682	Female	0		Demo	—	Menu
<input type="checkbox"/>	0445佩戴者 MT2	355932600100445	Male	0		Demo	—	Menu
<input type="checkbox"/>	0874佩戴者 MT2	355932600100874	Male	0		Demo	—	Menu
<input type="checkbox"/>	Senior Demo A		Male	59		Demo	—	Menu


3. Choose 'Add' to register a Target.

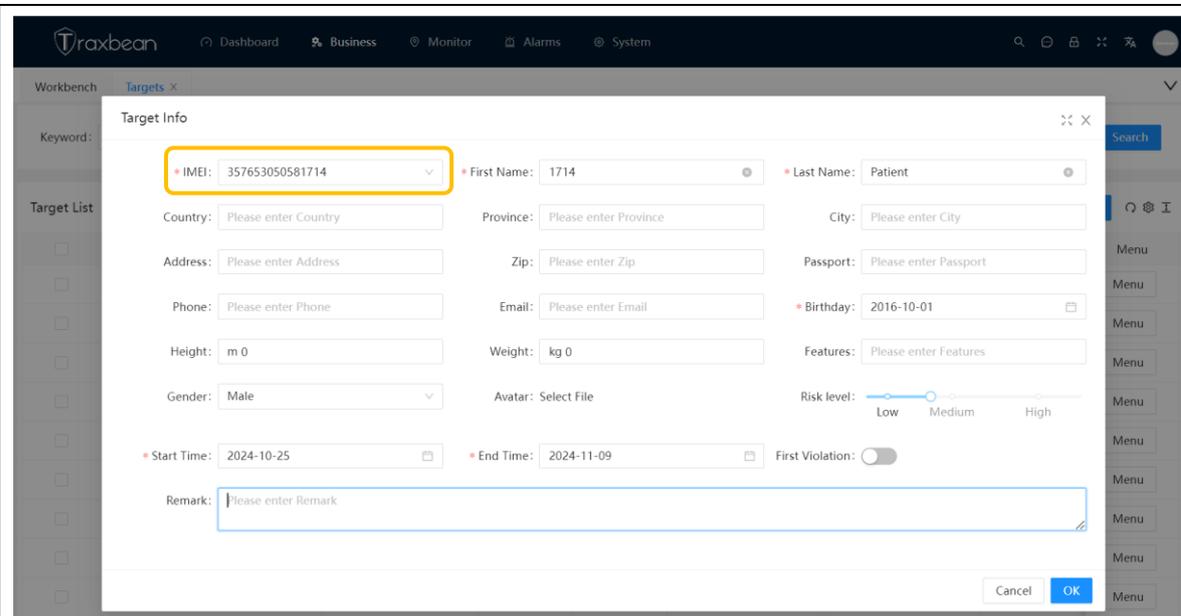


4. Input IMEI to band the devcie to the person.  
Input the personal information, monitoring start and end times.

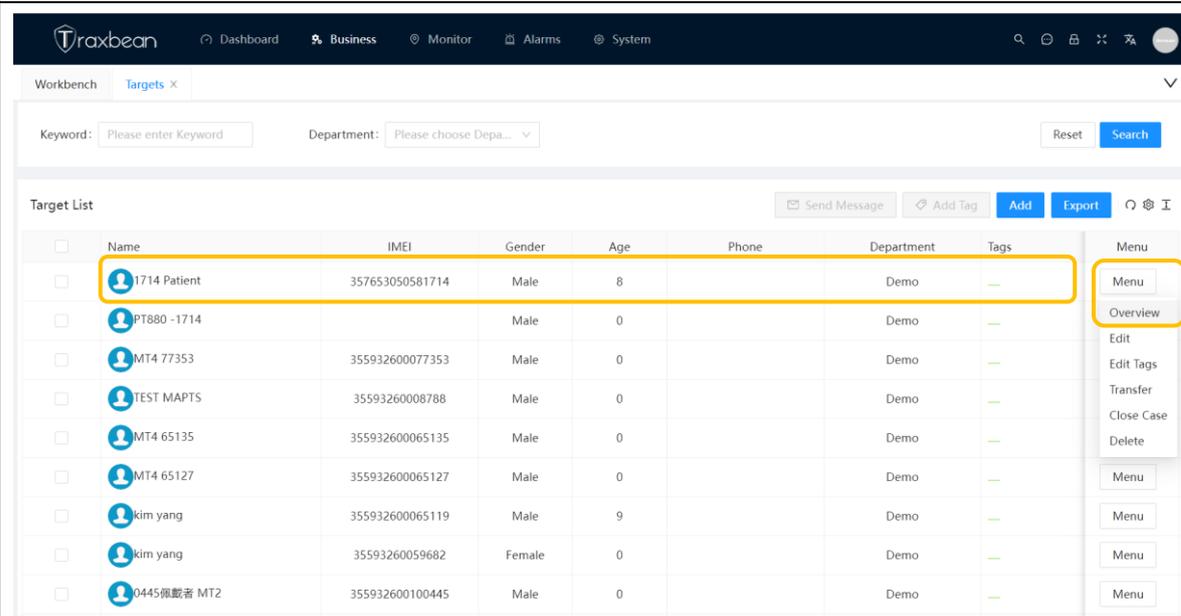


5. Select a target, choose 'Overview' in the Menu list.

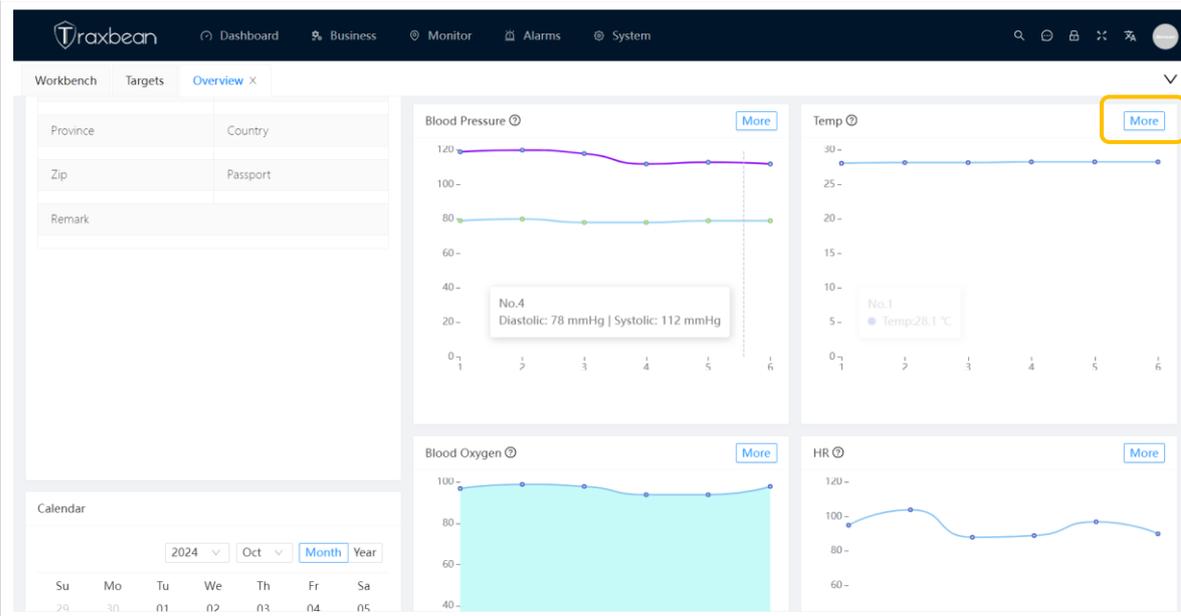

6. The overview page shows health reports such as blood pressure, temperature, blood oxygen, and heartrate, etc. Choose any one of them to check more details .

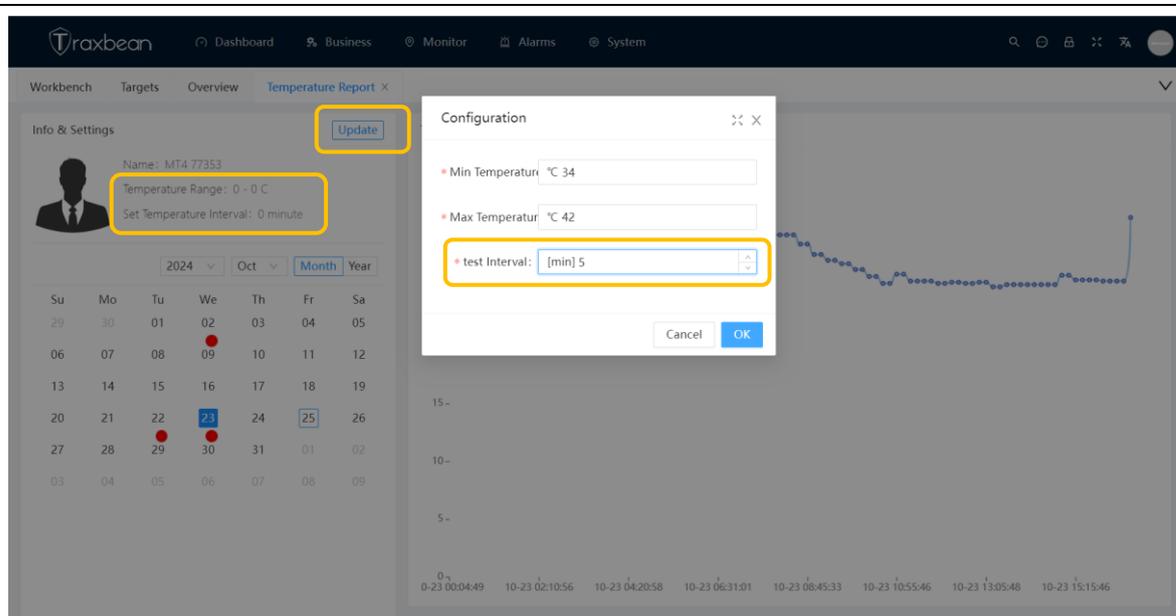

7. The health detail page shows details health data record and current detection parameters. Press 'Update' to configure the health detection parameters. The test interval is working for all kinds of health sensors such as blood pressure, temperature, blood oxygen, and heartrate, etc.

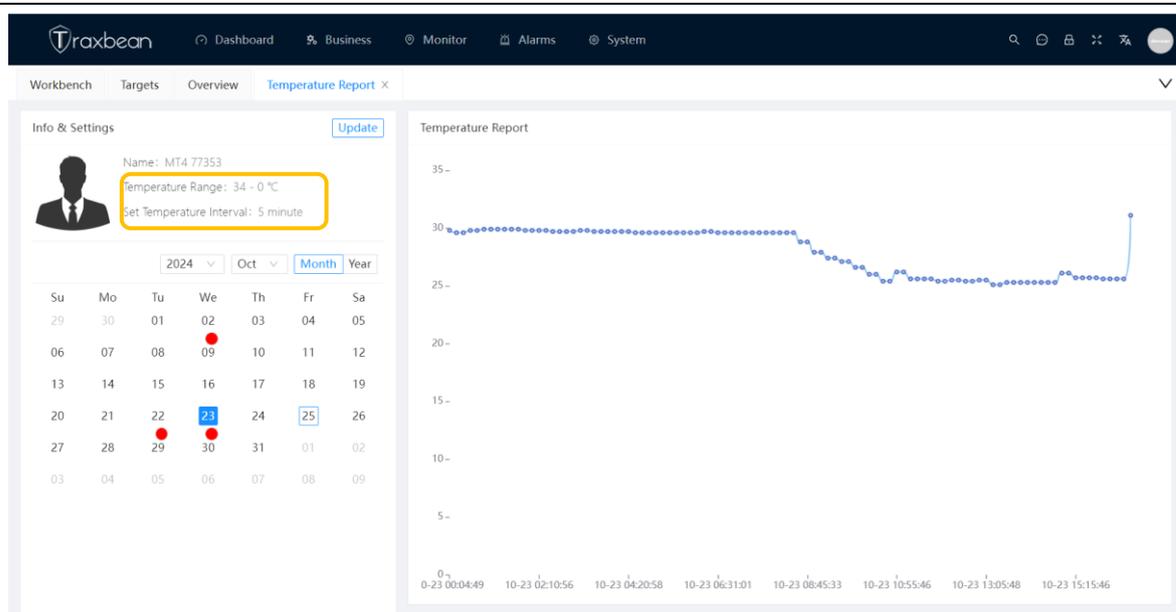

8. Considering the power consumption and sensor heating situation, we suggest that you set the health check interval to no less than 3 minutes. Press 'OK' to configure the health test interval then you will find the parameters changed. The device would follow the parameters.